








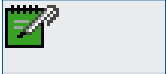

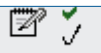
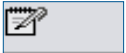



# Integrad

## Selecting a case

LENOX HILL RADIOLOGY  
& MEDICAL IMAGING ASSOCIATES P.C.

61 East 77th Street, New York, NY 10021  
TEL: 212-772-3111 • FAX: 212-772-0468  
www.lenoxhillradiology.com

	<b>Configuration tool.</b> Allows us to choose what fields we wish to have in our menu. We can change the caption of these fields if we wish. E.g. from accession # to acc no.
	The binoculars allow us to define our <b>search</b> . We choose what fields we want to filter by. Name, date of exam, modality type.
	<b>Save search as.</b> This allows us to create our own tabs. Right click on search results. Give the search a name e.g. All CT Scans.
	The scales allow us to <b>compare</b> two series. However, we must have two series checked to begin. 
	The arrows allow us to move between the <b>last and previous</b> pages of studies
	The arrows allow us to move between the <b>first and next</b> pages of studies.
	Shows how many studies found for search.
	We decide how many per page. Approx 16 will fill one screen.
	This notepad icon indicates that a report has been written to this study. Yellow= <b>note</b> Green = <b>Prelim report</b>
	Has been <b>reviewed</b> .
	Has been <b>reviewed and reported</b> .
	No report or note written yet, one can be written at this stage. You may write or amend your own notes but not one belonging to anyone else.
	Leave this at the defaulted value.

To **view** your patients images/reports online, **log on to [www.lenoxhillradiology.com](http://www.lenoxhillradiology.com)**.  
**Enter** Physicians & Medical Professionals. **Access** images or reports. **Enter** your username and password.







If you have any questions or comments regarding this service please contact  
Sinead Conway at (212) 772-3111, EXT: 1007, [sconway@lenrad.com](mailto:sconway@lenrad.com).







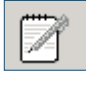



# Integrad

## Image display

**LENOX HILL RADIOLOGY  
& MEDICAL IMAGING ASSOCIATES P.C.**

61 East 77th Street, New York, NY 10021  
TEL: 212-772-3111 • FAX: 212-772-0468  
www.lenoxhillradiology.com

Left Click	To select a function you left click, then move to image and keep left click depressed to activate function.
Right Click	Right clicking on an image will give you your options and is another way to change window levels, page format etc
Spacebar	Allows you to toggle through the pre chosen windows, for that study.
Home, End	Allows immediate movement between first and last images.
	Select <b>"page/scroll"</b> then left click on image. You can scroll through images individually. Use can use the left click, the wheel or the up and down arrows.
	Contrast. Select <b>"window level"</b> then left click on image. You can adjust the window level for each image. Right click to choose another window level.
	Select <b>"zoom"</b> then hold down left mouse to alter zoom level for displayed image.
	Select <b>"pan"</b> or (hand) it allows us to drag image around the page to view area of interest.
	<b>"Select,"</b> allows us to mark an image as selected for future reference.
	This is the <b>navigator</b> and will only appear if all series are not displayed. It shows exactly how many series there are. Double click on image to display a series or image.

	This film view indicates how it's going to look printed.
	Magnifying glass. Zooms up a particular area and is best used in xray.
	Shows <b>cross references</b> and levels of slices.
	If two studies are chosen, they can be <b>synchronized</b> . We can then go through images on both studies at once.
	Create a copy
	The <b>overlay</b> with anatomical orientation can be removed if wished.
	Make or edit a <b>note</b> of your own. Also view other notes.
	<p>Dictated - Close Study            Dictated - Keep study Opened            Reviewed - Close Study            Reviewed - Keep Study Opened            Exit</p> <p>Options of how to mark exam.            If nothing has changed, simply choose exit.</p>
Calibrate image	Select this text option. Follow instructions from pop up. Draw a line using the mouse and the ruler on the right side. Enter how long this segment is. Choose to calibrate per image not series.
	 Type of measurements e.g. angles